

eUnity cheat sheet

Modes

| | |
|---|--|
| W | Toggle Window-Level Mode |
| Z | Toggle Pan / Zoom-Level Mode |
| N | Toggle Navigate Mode |
| T | Toggle Triangulation / Cursor Mode |
| L | Toggle Linear Measurement Mode |
| E | Toggle Elliptical ROI Measurement Mode |
| F | Toggle Free-form ROI Measurement Mode |
| A | Toggle Angle Measurement Mode |
| B | Toggle Cobb Angle Measurement Mode |
| C | Toggle Circle Measurement Mode |
| Y | Toggle Arrow Annotation Mode |
| G | Toggle Freehand Annotation Mode |
| Q | Toggle Series Repositioning Mode |
| V | Toggle Spine Labeling Mode |

Flips and rotations

| | |
|-----------|---------------------------------|
| Shift + C | Rotate Series Clockwise |
| Shift + W | Rotate Series Counter-clockwise |
| Shift + H | Series Horizontally |
| Shift + V | Flip Series Vertically |

Image/Series navigation tools

| | |
|------------|--|
| Up arrow | Scrolls images from a higher number image to a lower number image |
| Down arrow | Scrolls images from a lower number image to a higher number image |
| Home | Returns the active View Box to the first image (lowest number image) |
| End | Returns the active View Box to |

| | |
|-------------|---|
| Left arrow | the last image (highest number image) |
| Right arrow | Place the previous series, which is not currently displayed, into the active viewport |

| | |
|-------------|---|
| Right arrow | Place the next series, which is not currently displayed, into the active viewport |
|-------------|---|

Mouse actions

| | |
|---------------------------------|---|
| Mouse Right Drag: | Window Level Adjust |
| Mouse Middle Drag: | Zoom with Mouse |
| Mouse Left Drag: | > |
| Mouse Left Drag + Ctrl: | Temporarily unlink series and scroll images |
| Mouse Right Drag + Ctrl: | Pan with Mouse |
| Mouse Middle Drag + Ctrl: | Triangulate with Mouse |
| Mouse Left Drag + Ctrl + Shift: | Series reposition |
| Right click | Show Context Menu |

Study Tools

| | |
|-------|---|
| O | Revert study to original Window-Level and remove all markup |
| P | Toggle Cine Mode |
| D | Toggle Demographics |
| 0 | Revert study to original Window Level |
| S | Toggle Reference Lines |
| K | Toggle Link Series |
| R | Show Report (if available) |
| M | Toggle Show / Hide Markup |
| Space | Toggle Cine Play / Pause |

Window Level Presets

| | |
|---------|---------------------------|
| [1 - 9] | Apply Window Level preset |
|---------|---------------------------|

Image Sharpening presets

| | |
|-----------------|-------------------------------------|
| 0 | Revert sharpening level to original |
| Shift + [1 - 9] | Apply Window Level preset |
| Shift + S | Toggle Sharpening preset |

Zoom tools

| | |
|---|---------------------------------|
| + | Zoom in on the active viewport |
| - | Zoom out on the active viewport |

Markup

| | |
|-----|-----------------------------|
| Del | Deletes the selected markup |
|-----|-----------------------------|

MPR / 3D

| | |
|-----------|--|
| Shift + O | Create oblique |
| Shift + B | Create curved MPR |
| Shift + I | Toggle MPR curve orientation indicator |
| Shift + L | Toggle linked MPR curve orientation indicators |
| Shift + R | Reset MPR curve rotation angle |
| Shift + C | Toggle clip box |
| Shift + P | Cut plane mode |
| Shift + F | Scalpel mode |
| Esc | Cancel segmented area |
| Shift + W | Settings panel |

Last modification: 7/5/2020 8:50:24 AM

More information: defkey.com/eunity-client-outlook-shortcuts

[Customize this PDF...](#)